



online |
pedors247.com

phone |
800 750 6729

fax |
800 446 3101

In stock | ZERO Delamination | Easy to Heat & Fit | #1 Preferred Prefabs

Pedors® 3P Inserts



Available in full sizes from a women's size 5 up to a men's size 20
3 Pairs per pack, \$21 per pack or \$7 per pair. Samples available upon request.

The 3P insert has a 6mm durable 45A durometer polyurethane base layer for support with a 4mm cushioned 15A durometer cellular urethane Poron™ middle layer and a 4mm soft 25A durometer heat moldable polyethylene top layer that molds to the plantar surface of the foot.

Designed to provide support and moldability to ensure total contact with the plantar surface of the foot.

The 3P insert is sized to the Pedors last, but can also be fitted to some other depth shoes with some easy minor modifications if necessary. To fit the 3P insert, remove the insole or liner(s) that come with the shoe and using them as a template, trace around the bottom outside layer of the 3P insert. These trace lines will serve as your trim lines. Use a pair of heavy scissors to trim as necessary.

Heat moldable at 250 degrees F for 1 -2 minutes. Reviewed and meets requirements for coverage under the TSD.

Pedors® 2P Inserts



Available in full sizes from a women's size 6 up to a men's size 13
3 Pairs per pack, \$18 per pack or \$6 per pair. Samples available upon request.

The 2P insert is a has a 6mm durable 42-46 A durometer EVA closed cell base layer with a 4mm softer 25 A durometer EVA closed cell top cover.

Designed to provide support and moldability to ensure total contact with the plantar surface of the foot.

The 2P insert is sized to fit footwear with less depth and width than the Pedors Classic line. To fit the 2P insert remove the insole or liner(s) that come with the shoe and using them as a template, trace around the bottom outside layer of the 2P insert. These trace lines will serve as your trim lines. Use a pair of heavy scissors to trim as necessary.

Heat moldable at 250 degrees F for 1 -2 minutes. Reviewed and meets requirements for coverage under the TSD.



1-800-750-6729

www.pedors247.com

fax: 800-446-3101



GAL40M - Genext® Hiker Men - Bown with Black
GAL40W - Genext® Hiler Women - Gray with Red

Sizes: Women's 5 - 12, Men's 7 - 14
Widths: Medium, Wide, X-Wide
(Not Unisex - men's & women's lasts)

EXTRA DEPTH Accommodates deep insert modifications and foot deformities. Total Removable Depth - 9/16th"



GCV10 - Genext® Comfort Hook & Loop

Sizes: Women's 5 - 12, Men's 7 - 14
Widths: Medium, Wide, X-Wide
(Not Unisex - men's & women's lasts)

EXTRA DEPTH Accommodates deep insert modifications and foot deformities. Total Removable Depth - 9/16th"



GCL10 - Genext® Comfort Lace

Sizes: Women's 5 - 12, Men's 7 - 14
Widths: Medium, Wide, X-Wide
(Not Unisex - men's & women's lasts)

EXTRA DEPTH Accommodates deep insert modifications and foot deformities. Total Removable Depth - 9/16th"



GAL10 - Genext® Athletic Lace Black
GAL20 - Genext® Athletic Lace White

Sizes: Women's 5 - 12, Men's 7 - 14
Widths: Medium, Wide, X-Wide
(Not Unisex - men's & women's lasts)



GAV10 - Genext® Athletic Hook & Loop Black
GAV20 - Genext® Athletic Hook & Loop White

Sizes: Women's 5 - 12, Men's 7 - 14
Widths: Medium, Wide, X-Wide
(Not Unisex - separate men's & women's lasts)



BOS - Genext® Orthotic System (formerly Beats™)

Sizes: Women's 5 - 11, Men's 7 - 14
Variants: Posted Heel or Non-Posted Heel
Metatarsal Pad or No Metatarsal Pad

About Pedors®

Pedors Shoes is known as the original orthopedic stretch shoe company.

The brand Pedors reflects the collaboration between **Pedorthists** and **Orthotists** from around the world and the company's founders to design affordable and appropriate footwear for people with special footwear needs.



PEDORS CLASSIC MAX

Black Only (MX600)

25 Sizes

Widths: X-Wide, XX-Wide, XXX-Wide



PEDORS CLOGS

Black (701) Beige (700)

8 Sizes

Widths: Medium

Pedors Are Available From:

pedors®
Comfort Is No Mean Feet™

PEDORS SHOES



Indications For Use

- Swollen Feet
- Edema / Lymphedema
- Forefoot Deformities
 - Bunions / Corns
 - Hammertoes
- Diabetes / Arthritis
 - Fall Prevention
- Orthotics / Bracing

STRETCH COMFORT



Machine Washable

THE ORIGINAL PEDORS CLASSICS

Black (600), Beige (601), Brown (605)
25 Sizes
Widths: Medium, Wide, X-Wide



PEDORS STRETCH WALKERS

Black Only (800)
23 Sizes
Widths: Wide, X-Wide, XX-Wide

Pedoprene™ Uppers

All Pedors shoes feature a four-way stretch heat-moldable proprietary EVA material named Pedoprene™. This material is unique to the Pedors brand.



PEDORS MARY JANES

Black (500), Beige (501)
15 Sizes
Widths: Medium, Wide, X-Wide

Sizes & Widths

Pedors shoes are available in full and half sizes and in five widths covering 137 unique fittings. Your local Pedors authorized dealer is a qualified shoe fitting professional who will be able to size and fit you with an affordable "off-the-shelf" shoe that is as close to a custom fit as possible.



Machine Washable

PEDORS HIGH TOP

Black Only (600-H)
17 Sizes
Widths: XX-Wide



PEDORS 3P TRI-LAM INSERTS

Beige Only
15 Sizes (Sold in Packs of 3 Pairs)
Base - Polyurethane
Middle - Poron
Top - Polyethylene



Fantastically Functional Footwear™

1-800-750-6729
www.pedors247.com
fax 800-446-3101



Item #600 - The Classic in Black
Item #601 - The Classic in Beige

Sizes: Women's 3 - Men's 15
Widths: Medium, Wide, X-Wide

Applications:

Healing Shoe Critical Care
Nursing Homes Homebound



Item MX600 - The Classic MAX

Sizes: Women's 3 - Men's 15
Widths: 2E, 4E, 6E

Applications:

Healing Shoe Critical Care
Nursing Homes Homebound



Item #800 - Stretch Walker Black

Sizes: Women's 5 - Men's 15
Widths: Wide, X-Wide, XX-Wide

Applications:

Healing Shoe Critical Care
Nursing Homes Homebound



Item #500 - Mary Jane in Black
Item #501 - Mary Jane in Beige

Sizes: Women's 5 - 12
Widths: Medium, Wide, X-Wide

Applications:

Casual Accomodation
Light Edema, bunions, hammer-
toes, corns.



Item #600-H - The High Top

Sizes: Full sizes
Pediatric/Women's size 1 thru
Men's 17

Applications:

Wheelchair
Ankle support and control
Developmental Disabilities



Item 3Pins
Tri-Lam
Heat Moldable

Sizes: Full Sizes
Women's 5 -
Men's 20

Item 2Pins
Bi-Lam
Heat Moldable

Sizes: Full Sizes
Women's 6 -
Men's 13



Post-Op. Rehabilitate,
Regenerate, Recover.



Wound Care - Prevent,
Protect, Preserve.



Arthritis - these shoes accom-
modate painful arthritic feet and
also have easy closures for pain
free fastening.



Diabetes - The shoes have
seamless forefoot areas
that reduce rubbing and
abrasions and hence
ulceration.



Accommodates both internal
and external AFO Bracing.



Machine Washable - Many
Pedors® shoes are machine
washable - aiding with
incontinence clean-up.

All Shoes Feature:



All Pedors shoes feature Pedoprene™ uppers/panels.
eLEVate™ technology allows for spot heat molding when
necessary. This very stretchable material will accommodate
the most demanding foot with or without heat molding.



Removable foot-bed
for customized inserts
or orthotics.



Accommodates deep insert
modifications and foot deformities.



Pedors® shoes are easily
modifiable - e.g. rocker
sole, leg-length discrepancy,
wedges.



Pedors® shoes are
lightweight allowing for
modifications without
making the shoe too
heavy.



PEDORS SHOES REFERENCE CATALOG

THE ULTIMATE SHOES FOR THE HARD TO FIT FOOT

Pedors Quick Reference Guide

Highly Recommended For Condition / Use Recommended Not Recommended For Condition / Use

	3P Inserts	Genext Orthotics	Clogs	Mary Janes	Classics	Stretch Walkers	High Tops	Classic MAX	Classic Slide
Widths Available	N/A	N/A	M	M, W, XW	M, W, XW	W, XW, XXW	XXW	XW, XXW, XXXW	XXXW
Colors Available	Bone	Tan	Black, Beige, Brown	Black, Beige	Black, Beige, Brown	Black	Black	Black	Black
Very Swollen Feet / Edema Lymphedema									
Swollen Feet Extra-Wide Feet									
Diabetes Peripheral Neuropathy									
Arthritis PAD / PVD									
Heel Pain / Plantar Fasciitis Ball of Foot Pain /				With Orthotic	With Orthotic	With Orthotic			
Arch Support				With Orthotic	With Orthotic	With Orthotic	With 3P Inserts	With 3P Inserts	
Sore Feet Easy On And Off - Easy				With 2P Inserts	With 3P Inserts	With 3P Inserts			
Bunions / Hallux Valgus Toe Deformities /									
Bracing / AFOs Post Op / Wound Care									
Post-Op Wrapped & Bandaged Feet									

Pedors® Stretch Walkers (#800)



The Pedors Stretch Walker are extra depth shoes designed for people with swollen feet, hammertoes, forefoot problems and braces. They are also worn by people who have a need for very deep and wide shoes due to substantial orthotic devices, bandaging or other foot problems. Orthotic & Brace friendly.

- The Pedors Stretch Walkers Are Unisex Shoes For Both Men & Women
- **Widths & Sizes:** Wide, X-Wide, XX-Wide. Full & Half Sizes: Women's 5 to Men's 15
- Runs True To Size
- **Colors:** Black

Features & Benefits

- Four Way Stretch Pedoprene™ Panel
- Leather Backing Material
- Removable Flat EVA Insole
- Dual Hook & Loop Closure For Easy On & Off
- Multiple Removable EVA Liners Provide Double Extra Depth
- EVA Midsole With Rubber Outsole With Toe Spring
- Orthotic & Brace Friendly
- Swollen Feet / Extra-Wide Feet
- Diabetes / Peripheral Neuropathy
- Hammertoes / Toe Problems
- Bracing / AFO
- Bandaged Feet & Wrapped Feet

Pedors® Mary Janes (#500 Black, #501 Beige)



The Pedors Mary Jane features a Pedoprene forefoot upper which stretches to accommodate any forefoot problems such as bunions, hammertoes, corns and mildly swollen feet. It is also worn by women who want comfortable extra-wide shoes.

- The Pedors Mary Janes are women's shoes.
- **Widths & Sizes:** Medium, Wide & X-Wide. Full & Half Sizes: Women's 5 to 12.
- Runs half-a-size large due to double depth toe box.
- **Colors:** Black, Beige

Features & Benefits

- Four Way Stretch Pedoprene™ Upper
- Leather Backing Material
- Removable Flat EVA Insole
- Single Strap Hook & Loop Closure For Easy On & Off
- Very Lightweight Shoes Minimize Tiredness
- Orthotic Friendly
- EVA Sole
- **Double Extra Depth Toe Box**
- Mildly Swollen Feet
- Extra-Wide Feet
- Diabetes / Arthritis
- Peripheral Neuropathy PAD / PVD
- Bunions
- Hammertoes / Toe Problems

Pedors® High Tops (#600-H)



Whether you need a boot that has depth for your orthotics or a boot to accept your foot bracing or just a comfortable warm boot that can accommodate your swollen feet, the Pedors High Top is like nothing else out there.

- The Pedors High-Tops Are Unisex Shoes For Both Men & Women
- **Widths & Sizes:** XX-Wide Only. Full Sizes: Women's 1 to Men's 15
- For half-sizes, order up to the next full size.
- **Colors:** Black Only

Features & Benefits

- Four Way Stretch Pedoprene™ Panel
- Canvas Backing Material
- Removable Flat EVA Insole
- Dual Hook & Loop Closure For Easy On & Off
- Orthotic & Brace Friendly
- EVA Sole With Toe Spring
- Reinforced Bumper Material Around Toe Area
- Provides Added Protection For Wheelchair Footplate
- Swollen Feet / Extra-Wide Feet
- Diabetes / Peripheral Neuropathy
- Hammertoes
- Toe Problems
- Bracing / AFO
- Bandaged Feet & Wrapped Feet

Pedors® Classics (#600 Black, #601 Beige, #605 Brown)



The original stretch orthopedic shoes – The Pedors® Classics. A full length Pedoprene™ four way stretch upper on a lightweight mid-sole makes the The Pedors Classic a super-comfortable all day indoor and outdoor shoe. Very adjustable easy closure.

- The Pedors Classics Are Unisex Shoes For Both Men & Women
- **Widths & Sizes:** Medium, Wide & X-Wide. Full & Half Sizes: Women's 3 to Men's 15
- Runs True To Size
- **Colors:** Black, Beige, Brown

Features & Benefits

- Four Way Stretch Pedoprene™ Upper
- Lightweight & Durable Canvas Back
- Removable Flat EVA Insole
- Single Strap Hook & Loop Closure For Easy On & Off
- Very Lightweight Shoes Minimize Tiredness
- Orthotic Friendly
- Swollen Feet / Extra-Wide Feet
- Diabetes / Peripheral Neuropathy
- Arthritis PAD / PVD
- Bunions
- Hammertoes / Toe Problems

Pedors® Classic MAX (MX600)



The Pedors Classic MAX is our Pedors Classic Super-Sized. It is designed for people with very swollen feet due to Edema, Lymphedema or generally swollen feet and ankles. The dual strap closure ensure the shoes open up widely to allow the feet to placed into the shoes rather than forced in.

- The Pedors Classic MAX Are Unisex Shoes For Both Men & Women
- **Widths & Sizes:** X-Wide (2E), XX-Wide (4E), XXX-Wide (6E). Full & Half Sizes: Women's 3 to Men's 15. For Lymphedema order 1/2 size up from pre-lymphedema shoe size.
- **Colors:** Black

Features & Benefits

- Four Way Stretch Pedoprene™ Upper
- Lightweight & Durable Canvas Back
- Removable Flat EVA Insole
- Dual Hook & Loop Closure For Easy On & Off
- Very Lightweight Shoes Minimize Tiredness
- Orthotic Friendly
- EVA Sole With Toe Spring
- Very Swollen Feet / Edema / Lymphedema
- Extra-Wide Feet
- Diabetes / PAD / PVD
- Bunions / Toe Problems
- Bracing / AFO
- Bandaged Feet & Wrapped Feet

NEW!

Pedors® Classic Slide (SL600)



The most requested product is now here! A super-wide Pedors Slide. These slide are available in a 6E width and and have a wide, easy to close single strap. Designed for people with very swollen feet. The adjustable velcro strap allows to adjust for girth and to secure the slide to the foot.

- The Pedors Classic Slides Are Unisex Shoes For Both Men & Women
- **Widths & Sizes:** XXX-W (6E) Width Only. Full Sizes: Women's 7 to Men's 15
- Runs True To Size
- **Colors:** Black

Features & Benefits

- Four Way Stretch Pedoprene™ Upper
- Lightweight & Durable Canvas Back
- Removable Flat EVA Insole
- Wide Hook & Loop Closure For Easy On & Off
- Very Lightweight Shoes Minimize Tiredness
- EVA Sole With Toe Spring
- Very Swollen Feet / Edema / Lymphedema
- Diabetes / PAD / PVD
- Bunions / Toe Problems
- Bandaged Feet & Wrapped Feet

Pedors® Clogs (#700 Beige, #701 Black, #702 Brown)



The Euro Clog gives you the ability to enjoy the comforts of Pedoprene™ by just slipping on European Style Slide. Wonderful to have around the house or for short trips out. Corns and bunions are no match for the fantastically comfortable clog.

- The Pedors Clogs are for women
- **Widths & Sizes:** Full sizes, 5 thru 12. Medium width only
- For half-sizes order up to next full size
- **Colors:** Black, Beige & Brown

Features & Benefits

The clog is available in one width only (medium) which due to the stretchable upper can accommodate a medium through to a wide fitting.

- Machine Washable
- Pedoprene™ Upper
- Great for Comfort, Arthritis, Bunions
- Not Suitable For Swollen Feet
- Not For Use With Orthotics / Inserts.

Pedors® 3P Inserts (#3PIIns)



Our hugely popular Pedors 3P Inserts have been designed to ensure total contact with the bottom surface of the foot which is the key for comfort as it ensures even weight distribution and reduces pressure points that can cause pain or discomfort.

- The Pedors 3P Inserts are for men & women
- Women's 5 thru Men's 20
- Sold In Packs Of Three Pairs - a year's supply. Change them out of your shoes every 4 months

Features & Benefits

1. durable base layer for support
 2. middle layer for bounce and
 3. soft top layer that molds to your foot as you wear it
- Evenly distributes weight across the sole of your foot to eliminate pressure points that cause calluses
 - Medical Grade Accommodative Insert

Genext Active Orthotics (Bos)



In just a few days you could be slipping these orthotics into your shoes to chase away the pains associated with Plantar Fasciitis, (heel pain) Metatarsalgia (ball of foot pain), fallen arches, sore ankles, tired feet and so much more.

- Genext Active Orthotics Available For Men & Women
- Full sizes, women's 5 thru Men's 15

Features & Benefits

- Cork & Rubber Shell For Support & Moldability
- Top Cover Of Brushed Polyester To Wick Away Moisture
- Tri-Density Charcoal Foam Padding
- Posted Heel Option Available
- Metatarsal Pad Option Available
- Plantar Fasciitis
- Heel Pain
- Ball of Foot Pain
- Metatarsalgia
- Arch Support
- Stability
- Alignment
- Comfort



Comfortable Footwear
For Uncomfortable Feet



WHOLESALE

Questions? Call 800 750 6729

Int (+1) 770 218 8282

www.pedors247.com

support@pedors247.com

Pedors Classic

Eligible for billing
HCPCS Code
A5500

Designed For: Swollen feet, wider feet, forefoot deformities, diabetes, arthritis.

Removable Insole

Great for those using the Pedors® 3P Insoles, Genext® Active Orthotics or custom orthotics

Single Strap Touch Closure

Fantastic for adjusting the fit throughout the day

Pedoprene™ Stretch Uppers

Heat moldable, seamless stretch uppers accommodate diabetes, arthritis, bunions, and other forefoot deformities

Soft Toe-Box

Extra-depth soft toe-box limits rubbing & abrasions

Super Light-Weight

EVA Sole and canvas heel provide for a super light-weight pair of shoes

Toe Spring Reduces risk of trips and falls



Colors

Black (#600)
Beige (#601)
Brown (#605)



Widths

Medium (B/C)
Wide (D/E)
X-Wide (2E)



Sizes

Women's 3 - 17
Men's 7 - 15

Sizing Advice

- Order customer's usual shoe size
- For mild swelling go up one width
- For severe swelling please view the Pedors Classic MAX

Questions?

USA Tel: 800 750 6729 | International Call (+1) 770 218 8282

Email: support@pedors247.com

Pedors Classic MAX

Eligible for billing
HCPCS Code
A5500

Designed For: Venous insufficiency, Edema, Lymphedema, wrapped or bandaged feet, wound care, post op, recovery, healing shoe.

Removable Insole

Great for those using the Pedors® 3P Insoles, Genext® Active Orthotics or custom orthotics

Dual Strap Touch Closure

Fantastic for adjusting the fit throughout the day and making it easy to place feet in the shoes

Pedoprene™ Stretch Uppers

Heat moldable, seamless stretch uppers accommodate diabetes, arthritis, bunions, and other forefoot deformities

Soft Toe-Box

Extra-depth soft toe-box limits rubbing & abrasions

Super Light-Weight

EVA Sole and canvas heel provide for a super light-weight pair of shoes

Toe Spring Reduces risk of trips and falls



Colors

Black (#MX600)



Widths

X-W (2E)
XX-W (4E)
XXX-W (6E)



Sizes

Women's 3 - 17
Men's 7 - 15

Sizing Advice

Sizing Advice: Mild edema XW (2E) Significant Edema XXW (4E) Severe Edema/ Lymphedema / Wrapped or Bandaged Feet XXXW (6E) + full size larger than previous size prior to condition.

Questions?

USA Tel: 800 750 6729 | International Call (+1) 770 218 8282

Email: support@pedors247.com

Pedors Mary Jane

Eligible for billing
HCPCS Code
A5500

Designed For: Designed as a dress shoe for forefoot deformities and edema. Open face can accommodate severe edema.

Removable Insole

Great for those using the Pedors® 2P Insoles or Genext® Low Profile Orthotics or custom orthotics

Single Strap Touch Closure

Fantastic for adjusting the fit throughout the day

Pedoprene™ Stretch Uppers

Heat moldable, seamless stretch uppers accommodate diabetes, arthritis, bunions, and other forefoot deformities

Soft Toe-Box

Extra-depth soft toe-box limits rubbing & abrasions

Super Light-Weight Sole

Toe Spring Reduces risk of trips and falls



Colors

Black (#500)
Beige (#501)



Widths

Medium (B/C)
Wide (D/E)
X-Wide (2E)



Sizes

Women's 5 - 12

Sizing Advice

- Order customer's usual shoe size
- If the customer is experiencing swelling, order the X-Wide Width
- Not intended for people with very swollen feet or Lymphedema

Questions?

USA Tel: 800 750 6729 | International Call (+1) 770 218 8282

Email: support@pedors247.com

Pedors Classic Slide

Designed For: Very Swollen Feet, Edema, Lymphedema, Bunions, Hammertoes

Removable Insole

Great for those using the Pedors® 3P Insoles, or custom orthotics

Single Strap Touch Closure

Fantastic for adjusting the fit throughout the day

Pedoprene™ Stretch Uppers

Heat moldable, seamless stretch uppers accommodate diabetes, arthritis, bunions, and other forefoot deformities

Soft Toe-Box

Extra-depth soft toe-box limits rubbing & abrasions

Super Light-Weight

EVA Sole and canvas back combine for a super light-weight pair of shoes

Toe Spring Reduces risk of trips and falls



Colors

Black (#SL600)



Widths

XXX-Wide (6E)



Sizes

Women's 7 - 17
Men's 7 - 15

Sizing Advice

As this pair of shoes is typically ordered for Lymphedema or very swollen feet, we recommend ordering a full size larger than what you consider to be the customer's normal shoe size

Questions?

USA Tel: 800 750 6729 | International Call (+1) 770 218 8282

Email: support@pedors247.com

Pedors Stretch Walkers

Eligible for billing
HCPCS Code
A5500

Designed For: Accommodating AFO's, hammertoes, bracing, custom orthotics, edema. Removable spacers for additional depth

Removable Insole + 2 Spacers

Great for those using Pedors® 3P Insoles, bracing, or very deep custom orthotics!

Dual Strap Touch Closure

Fantastic for adjusting the fit throughout the day.

Pedoprene™ Stretch Panel

Great for hammertoes!

Leather Toe Bumper

Offers protection

Leather Upper

Provides greater stability & durability. Ideal for those wearing braces



Toe Spring Reduces risk of trips and falls



Closures

Touch Closure (#800)



Widths

Wide (D/E)
X-Wide (2E)
XX-Wide (4E)



Sizes

Women's 5 - 17
Men's 7 - 15

Sizing Advice

- Order your usual shoe size
- For mild swelling go up one width
- Heavy Swelling / Bracing order the XX-W (4E)

Questions?

USA Tel: 800 750 6729 | International Call (+1) 770 218 8282

Email: support@pedors247.com

Pedors MAX Wrap Sandal

Designed For: Wrapped or bandaged feet. Wound care, severe edema, lymphedema, post-op recovery shoe

Tapered Wrap Design

Ensures comfort at the ankle flex area

Pedoprene Stretch Wraps

With up to 4 inch adjustable additional volume

Touch Closure

Easy adjustment as feet swell during the day

Ankle Strap

Elasticated positional ankle strap

Improved!

More rubbery sole for improved grip!

Toe Spring

Reduces risk of trips and falls



Colors

Black (#WRX600)



Widths

XXX-Wide (6E)



Sizes

Women's 5 - 17
Men's 7 - 15

Sizing Advice

As this pair of shoes is typically ordered for Lymphedema or very swollen feet, we recommend ordering a full size larger than normal for the patient

Questions?

USA Tel: 800 750 6729 | International Call (+1) 770 218 8282

Email: support@pedors247.com

Pedors 3P Insoles / Inserts

Eligible for billing
HCPCS Code
A5510

Designed For: Tri-lam Heat Moldable Diabetic Insert also fitted for additional plantar comfort

Blue Middle Layer


1/16" (1.588 mm) 15A durometer cellular urethane middle layer for bounce and energy return

Beige Top Layer

2/16" (3.175 mm) soft 25A durometer heat moldable polyethylene top layer that molds to the plantar surface of the foot

Base Yellow Layer

3/16" (4.763 mm) durable 45A durometer polyurethane base layer for support

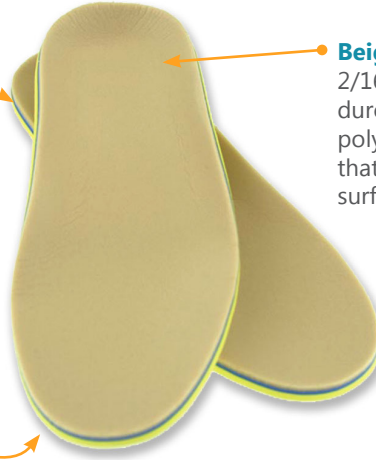

The 3P Insert are sized to our 4E last. Some trimming may be required for narrower widths. For 6E widths go up one size and trim if needed.



Sizing Advice

- Women's 5 - 22 | Men's 7 - 19
- Order your customer's usual shoe size
- These inserts work with all widths of Pedors

Do I have to heat mold them? Required for pre-fabricated Medicare diabetic insert reimbursement but optional for non-reimbursement. The inserts will mold to the shape of the customer's foot over a short period of time and will continue to get ever more comfortable

Questions?

USA Tel: 800 750 6729 | International Call (+1) 770 218 8282

Email: support@pedors247.com

Genext Active Orthotics

Designed For: Heel Pain (Plantar Fasciitis), Ball Of Foot Pain (Metatarsalgia), Alignment, Stability, Sore Feet

Two Heel Options

Posted: for flat feet or fallen arches
Neutral: for feet with normal arches

Cork and Rubber Interior Shell With Arch

For shape, support and alignment

Two Forefoot Options

With or without a metatarsal pad that helps to alleviate ball of foot pain

Foam Infused with Bamboo Charcoal

Controls moisture & odor

Polyester Top Cover

Wicks away moisture



Without Met Pad



With Met Pad



Sizes

Women's 5 - 12
Men's 7 - 15

Heel Options

Neutral: normal arch
Posted: flat feet or fallen arches

Forefoot Options

Neutral - normal
Met Pad - for ball of foot pain

Uses

Genext Active Orthotics are designed to help combat Plantar Fasciitis (Heel Pain) Metatarsalgia (Ball Of Foot Pain), Sore and Tired feet, alignment issues, fallen arches and much more.

Questions?

USA Tel: 800 750 6729 | International Call (+1) 770 218 8282

Email: support@pedors247.com

Pedors Euro Style Clog

Designed For: Women with bunions. Lightweight indoor / outdoor shoe

Pedoprene™ Stretch Uppers

Great for comfort, bunions, mild hammertoes, mild swelling, arthritis

Soft Toe-Box

Extra-depth soft-toe box limits rubbing & abrasions



Super Light-Weight Sole

Blown EVA Sole make for a super light-weight pair of mules

Colors

Black (#701)
Beige (#700)

Width

Medium Width
(stretches to a wide)

Sizes

Women's 5 - 12

Pedors 2P Inserts

Designed For: The Pedors Bi-laminate (2P) insert was developed to fit into non-extra-depth shoes where additional plantar support and cushioning is required.

Pink EVA Closed Cell Top Layer

4mm top cover with a 25A durometer for compression comfort

Yellow EVA Closed Cell Bottom Layer

6mm bottom cover with a 42-46A durometer for shape, stability & durability

Sizes

Women's 5 - 12



Sizes

Men's 7 - 13

Questions?

USA Tel: 800 750 6729 | International Call (+1) 770 218 8282

Email: support@pedors247.com

Pedors High-Tops

Eligible for billing
HCPCS Code
A5500

Designed For: Swollen Feet, Hammertoes, Sore Feet, Wheelchair, Extra Wide Feet, Deep Boots, Diabetes, Arthritis, Orthotic Shoes



Color
Black Only



Widths
XX-Wide (4E) Only



Sizes
Women's 1 - 17
Men's 7 - 15

Sizing Advice

Order your customer's usual shoe size

Special Feature

For a boot this functional one of the amazing things is how light it is!

Questions?

USA Tel: 800 750 6729 | International Call (+1) 770 218 8282

Email: support@pedors247.com

Pedors Wholesale Contact Information

Order online or offline;
by phone, fax or email
anytime.

pedors247.com

Sign-up, order online, view product information,
request marketing collateral, download fax order
forms and more.

Tel: 800 750 6729

+1(770) 218 8282

Fax: 800 446 3101

+1(770) 218 9027

support@pedors247.com

letswalk.com

Referral & Facility Locator Service